

# TD Talk User's Manual



## User's manual TD Talk

Version 1.1

06/2022

All rights reserved.

Copyright © Tobii Dynavox AB

No part of this document may be reproduced, stored in a retrieval system, or transmitted in any form, by any means (electronic, photocopying, recording, or otherwise) without the prior written permission of the publisher.

Copyright protection claimed includes all forms and matters of copyrightable material and information allowed by statutory or judicial law or hereafter granted, including without limitation, material generated from the software programs which are displayed on the screen such as screen displays, menus, etc.

The information contained in this document is proprietary to Tobii Dynavox. Any reproduction in part or whole without prior written authorization by Tobii Dynavox is prohibited.

Products that are referred to in this document may be either trademarks and/or registered trademarks of the respective owners. The publisher and the author make no claim to these trademarks.

While every precaution has been taken in the preparation of this document, the publisher and the author assume no responsibility for errors or omissions, or for damages resulting from the use of information contained in this document or from the use of programs and source code that may accompany it. In no event shall the publisher and the author be liable for any loss of profit or any other commercial damage caused or alleged to have been caused directly or indirectly by this document.

Content subject to change without notice.

Please check Tobii Dynavox website.

[www.TobiiDynavox.com](http://www.TobiiDynavox.com) for updated versions of this document.

### Contact Information:

Tobii Dynavox AB  
Karlsrovägen 2D  
182 53 Danderyd  
Sweden

+46 8 663 69 90

Tobii Dynavox LLC  
2100 Wharton Street, Suite 400  
Pittsburgh, PA 15203  
USA

+1-800-344-1778

Tobii Dynavox Ltd.  
Sheffield Technology Parks  
Cooper Buildings  
Arundel Street  
Sheffield S1 2NS  
United Kingdom

+46 8 663 69 90

TingDao Electronics Science & Technology (Suzhou) Co., LTD  
Unit 11/12, Floor 3, Building B, No.5 Xinghan Street, SIP, Suzhou  
P.R.China 215021


+86 512 69362880



# Table of Contents

1	Welcome .....	4
1.1	System Requirements .....	4
1.2	Supported Languages .....	4
1.3	Getting TD Suite .....	4
2	Resources and Support .....	5
2.1	Web Resources and Support .....	5
2.2	Learning Hub .....	5
3	Getting Started .....	6
3.1	Overview of TD Talk .....	6
3.2	Off-Screen Menu .....	7
3.2.1	How do I access the Off-Screen Menu in TD Talk? .....	7
3.3	Dashboard .....	8
3.3.1	How do I access the Dashboard in TD Talk? .....	8
3.3.2	How do I Exit and Close Down TD Talk? .....	8
3.4	Setting Up TD Talk .....	9
4	Using TD Talk .....	10
4.1	Using the Keyboard .....	10
4.2	Using Predictions .....	10
4.2.1	Word Predictions .....	10
4.2.2	Phrase Prediction .....	11
4.3	Editing a Message .....	12
4.3.1	How do I Edit a Message in TD Talk? .....	12
4.4	Using Quick Messages .....	12
4.4.1	How do I use Quick Messages in TD Talk? .....	12
4.4.2	How can I Edit the Quick Messages in TD Talk? .....	13
4.4.3	How can I Add a Quick Messages in TD Talk? .....	13
4.4.4	How can I Delete a Quick Messages in TD Talk? .....	14
4.5	Changing the Language .....	14
4.5.1	How can I Switch Speaking Languages in TD Talk? .....	14
5	TD Talk Settings .....	15
5.1	Activation .....	16
5.1.1	Buttons .....	16
5.2	Keyboard .....	18
5.2.1	Buttons .....	18
5.2.2	Prediction .....	19
5.2.3	General .....	20
5.3	Voice .....	21
5.3.1	Voices .....	22
5.3.2	Voice Banking Accounts .....	23
5.4	Device .....	25
5.4.1	Partner Window .....	25
5.5	General .....	27
5.5.1	How do I set my Application language in TD Talk? .....	27

# 1 Welcome

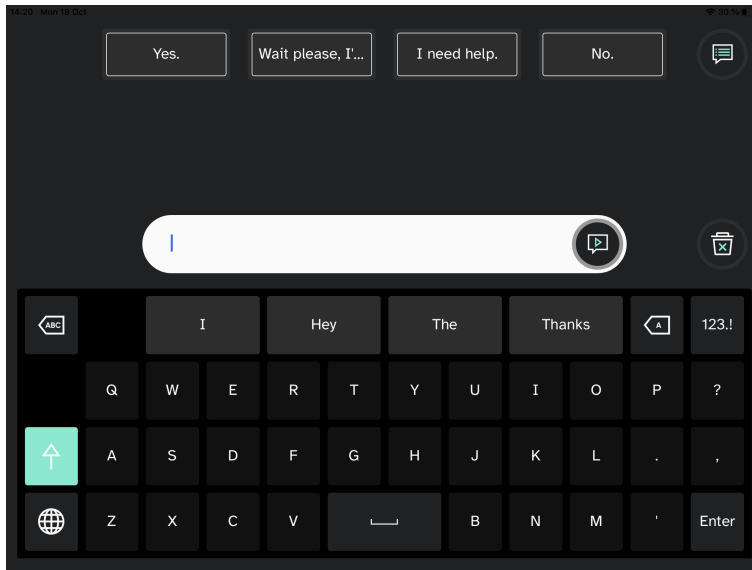

**Figure 1.1** TD Talk

TD Talk empowers people to communicate who they are and what they want to say, in the most fluid way possible. It is designed to make sentence construction as fast as possible. TD Talk enables individuals using it to remain an active part of the conversation. TD Talk looks and feels familiar so that people using it can get started in the shortest time possible. By learning a person's unique sentence patterns and word use, TD Talk continuously adapts to offer personally relevant options for communication.

## 1.1 System Requirements

Table 1.1 iOS

	Operating System	Device	Memory	Touch	Camera
<b>Minimum</b>	iOS 15.1	iPad 7th generation	4 GB	Integrated touch	Integrated camera
<b>Recommended</b>	iOS 15.1	iPad 8th generation	4 GB	Integrated touch	Integrated camera

## 1.2 Supported Languages

- Danish (DK)
- Dutch (NL)
- English (US)
- French (FR)
- German (DE)
- Italian (IT)
- Norwegian (NO)
- Spanish (Latin America)
- Swedish (SE)
- More languages coming soon!

## 1.3 Getting TD Suite

TD Talk is available through the Apple App Store for iPadOS.

## 2 Resources and Support

Tobii Dynavox offers several free resources to support and supplement your experience with TD Talk. Don't miss out on these tools that will help you get the most out of your Tobii Dynavox software!

### 2.1 Web Resources and Support

If you need support, your first stop should be the Training & Support section of the Tobii Dynavox website.

### 2.2 Learning Hub



The Learning Hub is currently English-only.

Visit [learn.tobiidynavox.com](https://learn.tobiidynavox.com) for documentation, training, and other resources, including:

- answers to common questions
- training videos
- recorded webinars
- online courses

# 3 Getting Started

If TD Talk is going to be used together with an Eye tracker, please make sure the to have an active calibration for the Eye tracker.

- iOS — use the TD CoPilot app for setting up the Eye tracker.  
For more information, see the TD Pilot User's manual.

## 3.1 Overview of TD Talk


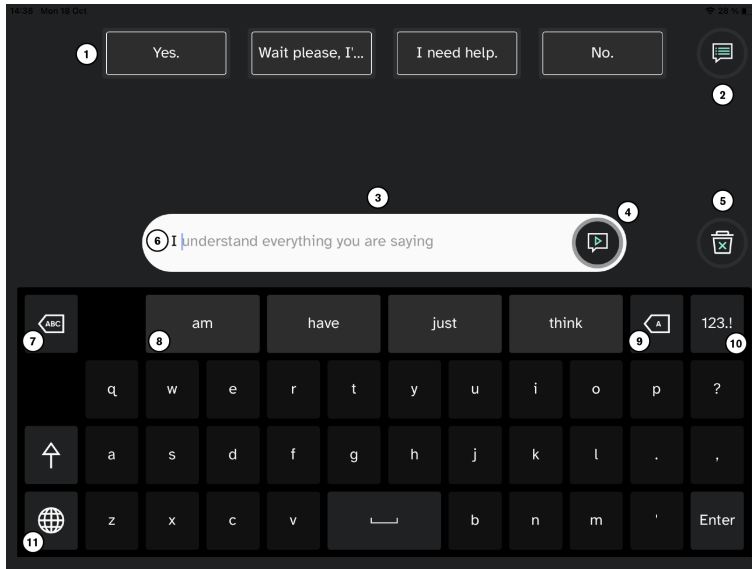


Figure 3.1 TD Talk

Position	Description	Position	Description
1	<b>Quick Messages</b> Pre-written messages that speak immediately when selected	7	<b>Delete Word</b> Delete the word to the left of the cursor.
2	<b>More Quick Messages</b> Open the Quick Messages page to speak or edit your Quick Messages.	8	<b>Word Prediction</b> Automatically predicts words based on what you've typed so far.
3	<b>Message Window</b> View and edit the text you would like to speak here.	9	<b>Backspace</b> Delete the character to the left of the cursor.
4	<b>Speak</b> Speak the text in the Message Window. You may also pause speech and resume speaking.	10	<b>Numbers Page</b> Access to numbers and special characters.
5	<b>Clear</b> Clear all text in the Message Window.	11	<b>Change Language</b> Switch between the languages you speak.
6	<b>Phrase Prediction</b> TD Talk learns from what you type and uses that information to predict phrases. Prediction is based on frequency and can make communication faster and more personal.		

## 3.2 Off-Screen Menu

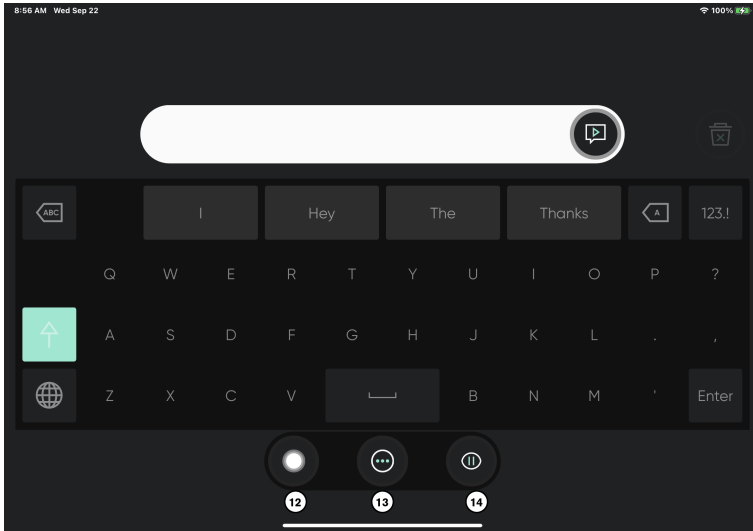

**Figure 3.2** TD Talk Off-Screen Menu


Position	Description	Position	Description
12	<b>AssistiveTouch</b> Turn on AssistiveTouch. Use AssistiveTouch to access the Home Screen and all of your other apps.	14	<b>Pause/Resume Eye Gaze</b> Toggle button to temporarily pause and resume eye gaze.
13	<b>More</b> Access the Dashboard and Settings.		

### 3.2.1 How do I access the Off-Screen Menu in TD Talk?

There are two ways of accessing the Off-Screen Menu in TD Talk:

- **As a user:** Fixate your gaze below the screen, in the middle of the eye tracker or the area of the eye tracker.
- **As a caregiver:** Swipe up from the letter **T** on the TD Talk keyboard.

## 3.3 Dashboard

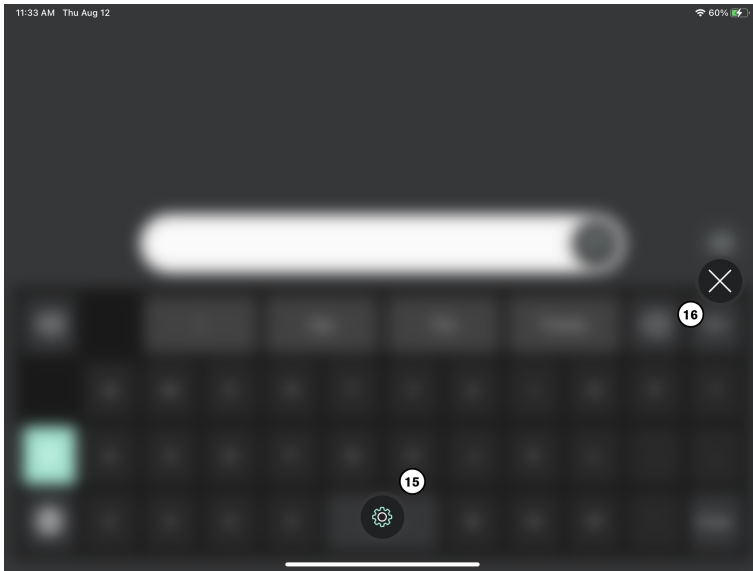


**Figure 3.3** TD Talk Dashboard

Position	Description	Position	Description
15	<b>TD Talk Settings</b> For more information, see <i>5 TD Talk Settings</i>	16	<b>Exit the Dashboard</b> Select the button to exit the Dashboard..

### 3.3.1 How do I access the Dashboard in TD Talk?


- Access the settings in one of the following ways:
  - As a user:** Fixate your gaze below the screen, in the middle of the eye tracker or the area of the eye tracker.
  - As a caregiver:** Swipe up from the letter **T** on the TD Talk keyboard.



The  (More) button will become visible in the lower middle part of the screen.

- Select the  (More) button to open the More.


### 3.3.2 How do I Exit and Close Down TD Talk?


- Access the settings in one of the following ways:
  - As a user:** Fixate your gaze below the screen, in the middle of the eye tracker or the area of the eye tracker.
  - As a caregiver:** Swipe up from the letter **T** on the TD Talk keyboard.







The  (AssistiveTouch) button will become visible in the lower middle part of the screen.

- Select the  (AssistiveTouch) button to open the AssistiveTouch Menu.
- Select the  (Home) button from the AssistiveTouch Menu.

## 3.4 Setting Up TD Talk

1. Select the  TD Talk app on the Home screen of the iPadOS device.
2. Access the settings in one of the following ways:
  - **As a user:** Fixate your gaze below the screen, in the middle of the eye tracker or the area of the eye tracker.
  - **As a caregiver:** Swipe up from the letter T on the TD Talk keyboard.

The  (More) button will become visible in the lower middle part of the screen.

3. Select the  (More) button to open the More.
4. Select the  (Settings) action to open the TD Talk Settings.
5. Set the TD Talk application language:
  - a. Select the  (Next Screen) button on the right-hand side.
  - b. Look for the  (General) icon to find the correct settings card.
  - c. Select the **Open** button.
  - d. Look in the **Application Language** section.
  - e. Select the radio button next to the language you want the application to be in.
  - f. Select the  (Back) button to go back to TD Talk Settings.
6. Select the voice(s) to use:
  - a. Look for the  (Voice) icon to find the correct settings card.
  - b. Select the **Open** button.
  - c. Look in the **Your Voice/s** section.
  - d. Select the radio button under the installed language you want to speak.



If the wanted language is missing, see 5.2.3.1 *How do I Add/Download a New Language/Keyboard Language in TD Talk?*, page 20 and 5.2.3.2 *How do I select and activate the Keyboard Languages to use/be available in TD Talk?*, page 21.

- e. Select the radio button next to the voice to use in the list of voices available for the language to speak. Use the “Scroll up” and Scroll Down” buttons on the right to scroll through the list of voices available for the language.




To try out a voice, select the  next to the voice. If available.




If there is a  (download) instead of a radio button, select the button to download the voice.


- f. Repeat steps **d** and **e** if you want to be able to speak in more than one language.

7. Select the  (Close) button to exit TD Talk Settings.



In TD Talk you can quickly switch languages between the selected ones by selecting the  button in the lower left corner of the keyboard.

## 4 Using TD Talk

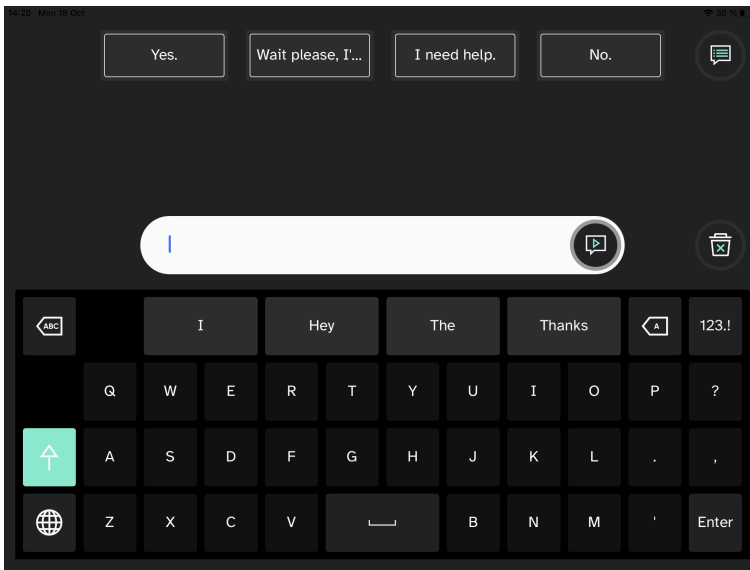

**Figure 4.1** TD Talk

### 4.1 Using the Keyboard

By selecting letters on the keyboard, words and sentences are formulated (or built) in the message window. Select the



(Speak) button when ready to speak.


### 4.2 Using Predictions

TD Talk offers both Word Prediction and Phrase Prediction which may decrease effort and/or increase speed of producing messages.

#### 4.2.1 Word Predictions

The word prediction engine used in TD Talk is from SwiftKey™ and is commonly used in tablets and smartphones. SwiftKey™ uses a blend of artificial intelligence technologies to predict the next word the user intends to type. SwiftKey™

learns from previous texts the user has written and offers predictions or presents predictions based on the current input text and what it has learned. The Word Prediction language will change depending on the Keyboard Language setting.

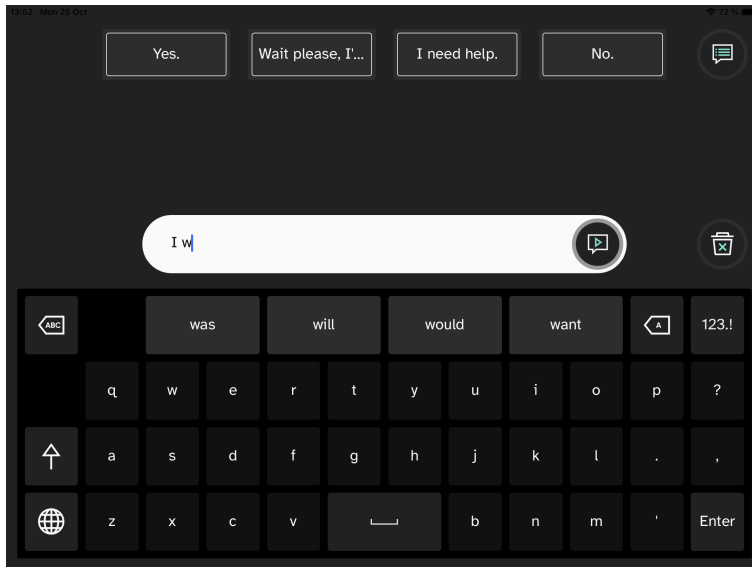


When Word Prediction is in use, four (4) boxes will appear above the Keyboard. The leftmost box will display the most probable predicted word, and the remaining three (3) boxes will contain one predicted word each. To select a predicted word, select the box with the correct word in it. If you can't find the word you are looking for, just keep on typing, and the word you are looking for may appear after the next character(s).

If you are using Word Prediction and want to type a word containing an accent, like café you will save time and effort by letting Word Prediction handle the accent. Simply type "cafe", and you will get a suggestion for "café".

#### 4.2.2 Phrase Prediction

TD Talk learns and analyses how you speak over time. Word prediction learns the words you say. Phrase prediction learns how you put those words together forming a personal phrase set. When you start typing, predicted phrases from your phrase set will be offered in the Message Window in grey text.

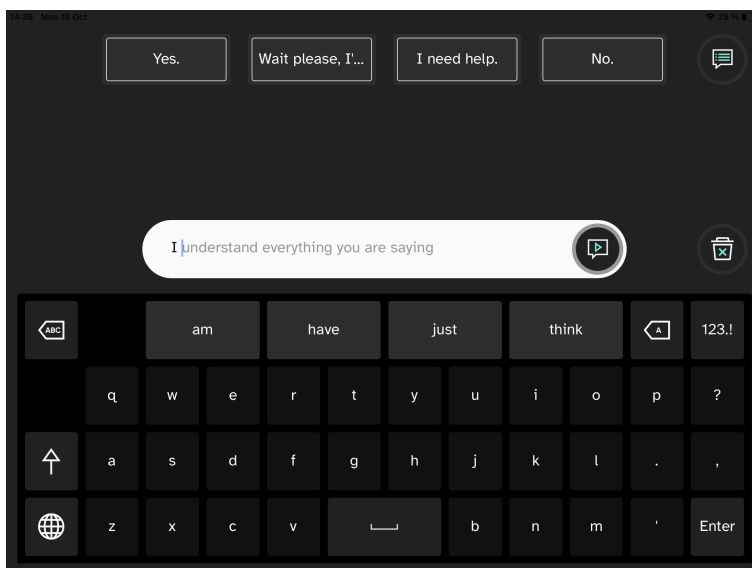


Look and fixate the gaze on the last word in the predicted phrase to select the phrase. The color on the phrase will change from grey to black indicating that it has been selected. it can now be spoken or edited.

## 4.3 Editing a Message


### 4.3.1 How do I Edit a Message in TD Talk?

1. Gaze at the section of the message to edit in the Message Window. The Edit Menu will open.

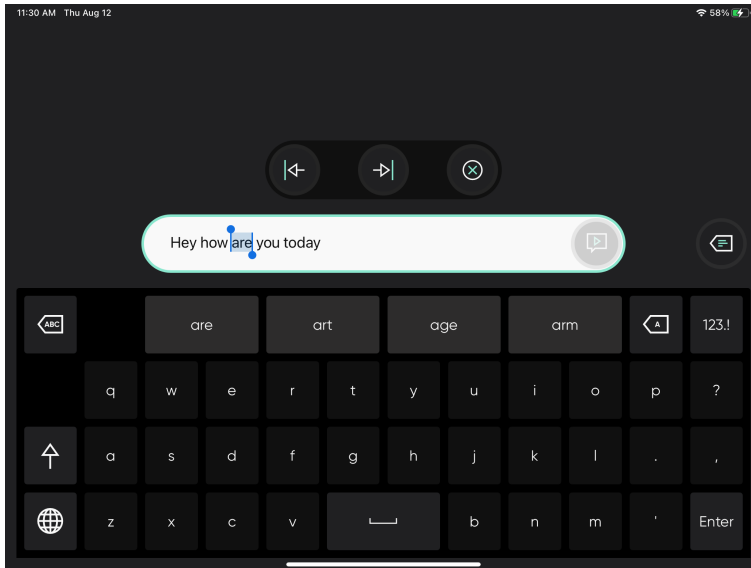

2. When a word is selected, type to replace the word, select a word from word prediction or select delete.


Use the  (Move Left) and  (Move Right) arrows if the cursor is in the wrong place.

3. Select the  (Close) button to exit Edit mode.

## 4.4 Using Quick Messages

### 4.4.1 How do I use Quick Messages in TD Talk?


There are two (2) types of Quick messages, the first ones are always visible on the Home screen. The second ones are


located under the  (More Quick Messages) button.



#### Quick Messages on the Home Screen

1. Select the **Quick Message** to speak.  
The selected message will be spoken immediately without being added to the Message Window and interfering with any message you have in process.



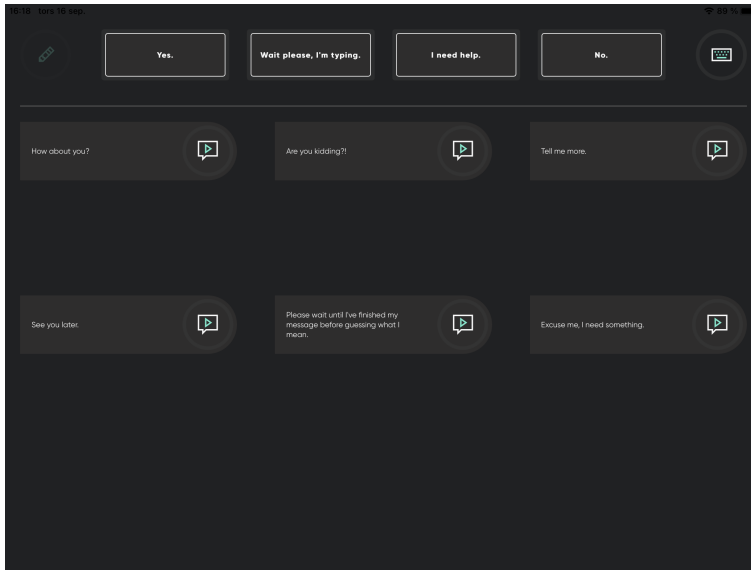



## More Quick Messages

1. Select the  (More Quick Messages) button.









2. Select the  (Speak) button next to the message.
3. Select the  (Keyboard) button to return to the Home screen.

### 4.4.2 How can I Edit the Quick Messages in TD Talk?








1. Select the  (More Quick Messages) button.
2. Select the  (Edit) button.
3. Select the  (Edit) button next to the message to edit.
4. Make the changes to the message with the keyboard.
5. Select the  (Save) button to save the changes to the Quick Message.
6. Select the  (Close) button to leave the editing view.
7. Select the  (Keyboard) button to return to the Home screen.

### 4.4.3 How can I Add a Quick Messages in TD Talk?

1. Select the  (More Quick Messages) button.
2. Select the  (Edit) button.
3. Select the  (Add) button next to the Add New location..
4. Write the new message with the keyboard.



5. Select the  (Save) button to save the changes to the Quick Message.
6. Select the  (Close) button to leave the editing view.
7. Select the  (Keyboard) button to return to the Home screen.

#### 4.4.4 How can I Delete a Quick Messages in TD Talk?


1. Select the  (More Quick Messages) button.
2. Select the  (Edit) button.
3. Select the  (Edit) button next to the message to delete.
4. Select the  Delete Message button.
5. Select the  (Save) button to save the changes to the Quick Message.
6. Select the  (Close) button to leave the editing view.
7. Select the  (Keyboard) button to return to the Home screen.






## 4.5 Changing the Language

### 4.5.1 How can I Switch Speaking Languages in TD Talk?

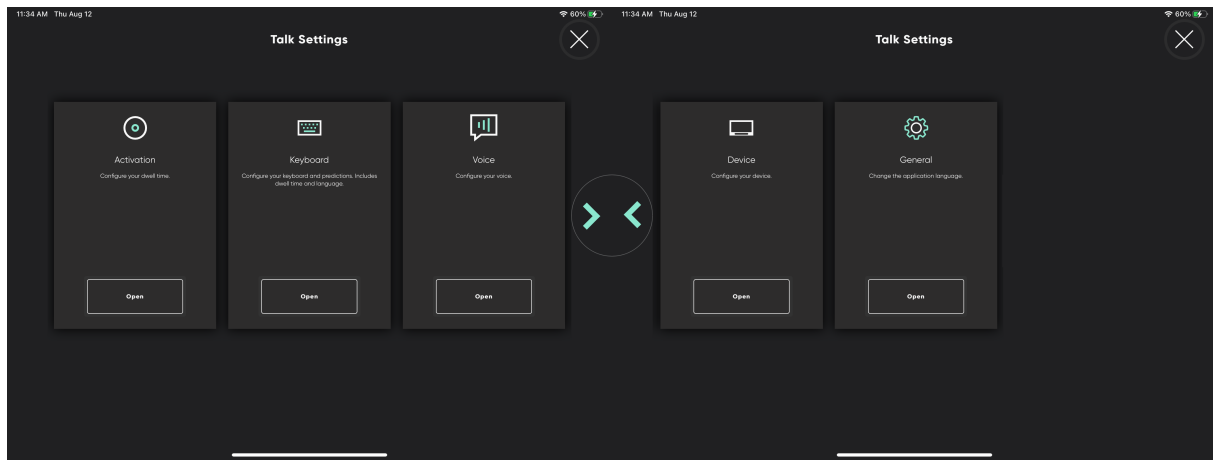1. Select the  (Language) button on the keyboard.
2. Select the language to speak  
If the wanted language is missing, see 5.2.3.1 *How do I Add/Download a New Language/Keyboard Language in TD Talk?*, page 20
3. Select the  (Back) button to go back

## 5 TD Talk Settings




Icons	Action	Description
	Activation	Only available if an Eye Tracker is installed and is activated. Opens the <b>Activation</b> settings. For more information, see <i>5.1 Activation</i> , page 16.
	Keyboard	Opens the <b>Keyboard</b> settings. For more information, see <i>5.2 Keyboard</i> , page 18.
	Voice	Opens the <b>Voice</b> Settings. For more information, see <i>5.3 Voice</i> , page 21.
	Device	Only available on the TD Pilot. Opens the <b>Device</b> settings. For more information, see <i>5.4 Device</i> , page 25.
	General	Opens the <b>General</b> settings,. For more information, see <i>5.5 General</i> , page 27.

## 5.1 Activation



Only available if an Eye Tracker is installed and is activated.

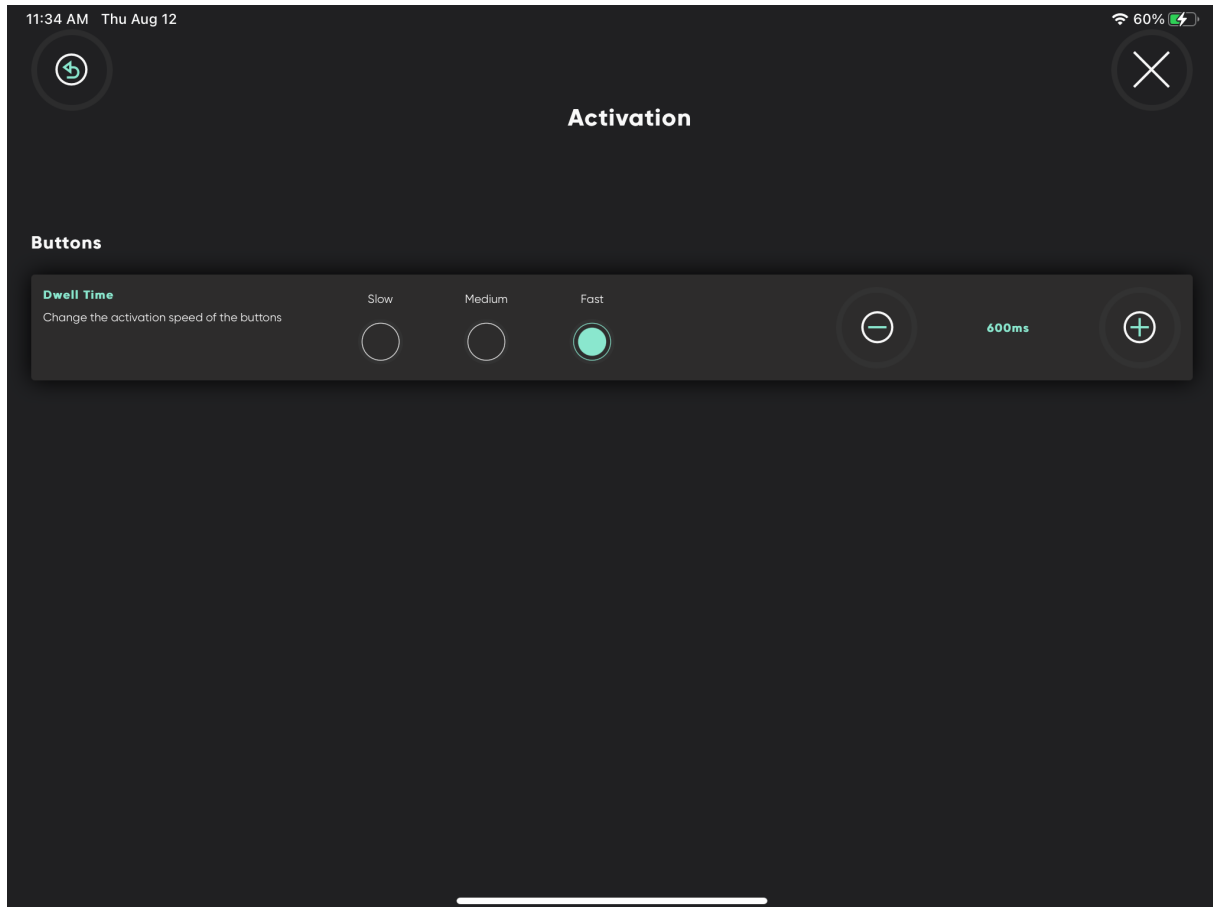

### 5.1.1 Buttons

#### 5.1.1.1 How do I Change the Dwell Time for Activating Buttons in TD Talk?



Buttons does not refer to those on the keyboard but the ones located in all other places in TD Talk.

1. Fixate your gaze below the screen, in the middle of the eye tracker or the area of the eye tracker.



The (More) button will become visible in the lower middle part of the screen.

2. Select the (More) button to open the More.



3. Select the (Settings) action to open the TD Talk Settings.



4. Look for the (Activation) icon to find the correct settings card.





5. Select the **Open** button.

6. Look in the **Buttons** section.

7. Select the radio button for the **Dwell Time** for one of the following values:


- Slow
- **Medium** (default)
- Fast


8. Select the  (Back) button to go back to TD Talk Settings.


9. Select the  (Close) button to exit TD Talk Settings.


#### 5.1.1.2 How do I Fine Tune the Dwell Time for Activating Buttons in TD Talk?

1. Fixate your gaze below the screen, in the middle of the eye tracker or the area of the eye tracker.

The  (More) button will become visible in the lower middle part of the screen.

2. Select the  (More) button to open the More.



3. Select the  (Settings) action to open the TD Talk Settings.

4. Look for the  (Activation) icon to find the correct settings card.


5. Select the **Open** button.


6. Look in the **Buttons** section.

7. Select one of the following buttons to fine tune the **Dwell Time**:


-  — to lower the Dwell Time
-  — to increase the Dwell Time

The value can be change between 200 ms — infinitive

8. Select the  (Back) button to go back to TD Talk Settings.

9. Select the  (Close) button to exit TD Talk Settings.

## 5.2 Keyboard

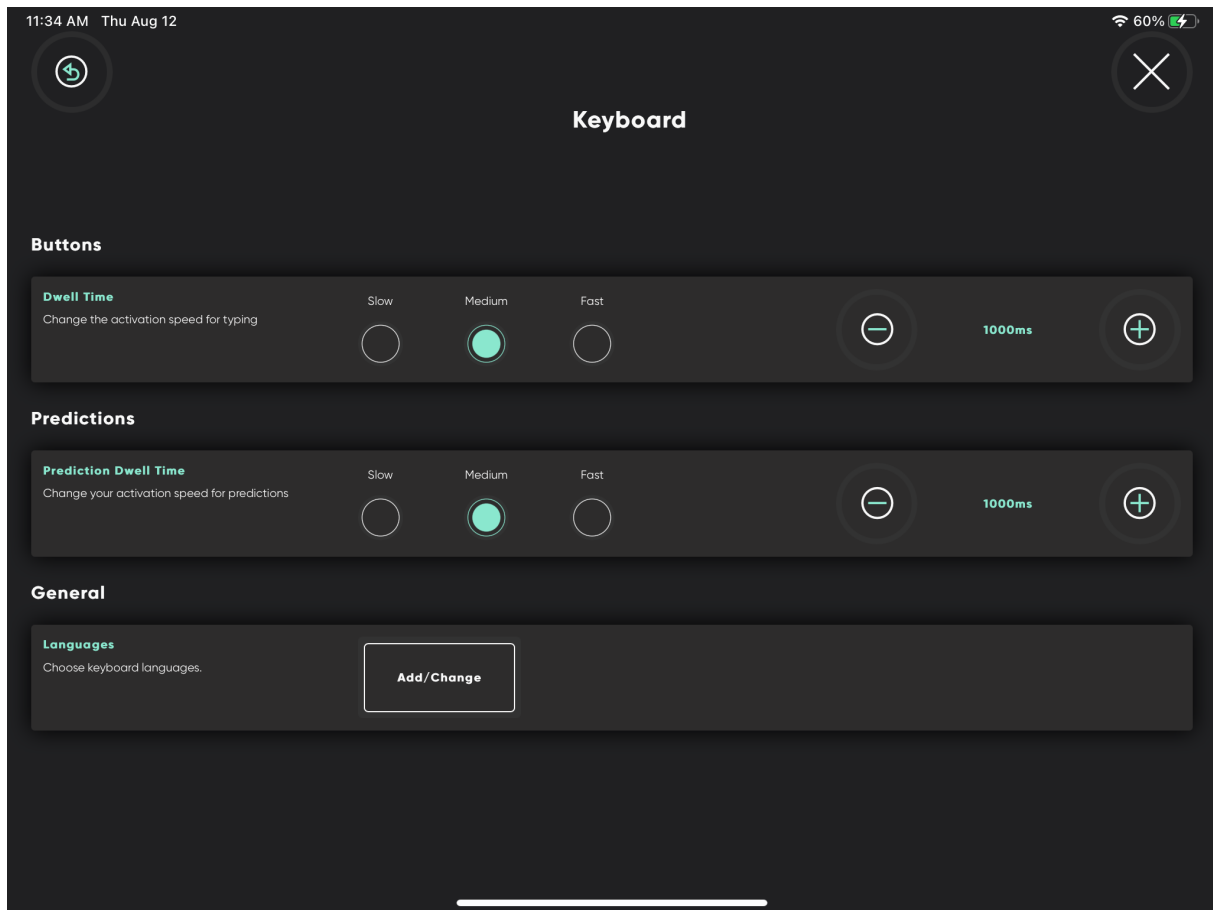






### 5.2.1 Buttons

#### 5.2.1.1 How do I Change the Dwell Time for Keyboard Buttons in TD Talk?

1. Fixate your gaze below the screen, in the middle of the eye tracker or the area of the eye tracker.

The  (More) button will become visible in the lower middle part of the screen.




2. Select the  (More) button to open the More.
3. Select the  (Settings) action to open the TD Talk Settings.
4. Look for the  (Keyboard) icon to find the correct settings card.
5. Select the **Open** button.
6. Look in the **Buttons** section.
7. Select the radio button for the **Dwell Time** for one of the following values:
  - Slow
  - **Medium** (default)
  - Fast



8. Select the  (Back) button to go back to TD Talk Settings.
9. Select the  (Close) button to exit TD Talk Settings.

### 5.2.1.2 How do I Fine Tune the Dwell Time for Activating Buttons in TD Talk?



1. Fixate your gaze below the screen, in the middle of the eye tracker or the area of the eye tracker.

The  (More) button will become visible in the lower middle part of the screen.

2. Select the  (More) button to open the More.
3. Select the  (Settings) action to open the TD Talk Settings.
4. Look for the  (Keyboard) icon to find the correct settings card.
5. Select the **Open** button.
6. Look in the **Buttons** section.
7. Select one of the following radio buttons to fine tune the **Dwell Time**:

-  — to lower the Dwell Time
-  — to increase the Dwell Time

The value can be change between 200 ms — infinite




8. Select the  (Back) button to go back to TD Talk Settings.
9. Select the  (Close) button to exit TD Talk Settings.


## 5.2.2 Prediction


### 5.2.2.1 How do I Change the Dwell Time for Prediction Buttons in TD Talk?

1. Fixate your gaze below the screen, in the middle of the eye tracker or the area of the eye tracker.

The  (More) button will become visible in the lower middle part of the screen.


2. Select the  (More) button to open the More.
3. Select the  (Settings) action to open the TD Talk Settings.
4. Look for the  (Keyboard) icon to find the correct settings card.
5. Select the **Open** button.
6. Look in the **Prediction** section.
7. Select the radio button for the **Dwell Time** for one of the following values:
  - Slow
  - **Medium** (default)
  - Fast




8. Select the  (Back) button to go back to TD Talk Settings.



9. Select the  (Close) button to exit TD Talk Settings.

### 5.2.2.2 How do I Fine Tune the Dwell Time for Prediction Buttons in TD Talk?



1. Fixate your gaze below the screen, in the middle of the eye tracker or the area of the eye tracker.

The  (More) button will become visible in the lower middle part of the screen.

2. Select the  (More) button to open the More.
3. Select the  (Settings) action to open the TD Talk Settings.
4. Look for the  (Keyboard) icon to find the correct settings card.
5. Select the **Open** button.
6. Look in the **Prediction** section.
7. Select one of the following radio buttons to fine tune the **Dwell Time**:

-  — to lower the Dwell Time
-  — to increase the Dwell Time


The value can be change between 200 ms — infinitive






8. Select the  (Back) button to go back to TD Talk Settings.
9. Select the  (Close) button to exit TD Talk Settings.


## 5.2.3 General

### 5.2.3.1 How do I Add/Download a New Language/Keyboard Language in TD Talk?

1. Fixate your gaze below the screen, in the middle of the eye tracker or the area of the eye tracker.


The  (More) button will become visible in the lower middle part of the screen.




2. Select the  (More) button to open the More.
3. Select the  (Settings) action to open the TD Talk Settings.
4. Look for the  (Keyboard) icon to find the correct settings card.
5. Select the **Open** button.
6. Look in the **General** section.
7. Select the “Choose keyboard language” **Add/Change** button.
8. Select the **Download** tab.
9. Select the  (Download) button next to the language to start the download of the new keyboard.
10. Select the **Download** button to start the download or the **Cancel** button to cancel the download.
11. Select the **Installed** tab.
12. Select the radio button next to the languages to activate the language.
13. Select the  (Back) button to go back to TD Talk Settings.


14. Select the  (Close) button to exit TD Talk Settings.



### 5.2.3.2 How do I select and activate the Keyboard Languages to use/be available in TD Talk?

1. Fixate your gaze below the screen, in the middle of the eye tracker or the area of the eye tracker.

The  (More) button will become visible in the lower middle part of the screen.


2. Select the  (More) button to open the More.
3. Select the  (Settings) action to open the TD Talk Settings.
4. Look for the  (Keyboard) icon to find the correct settings card.
5. Select the **Open** button.
6. Select the “Choose keyboard language” **Add/Change** button.
7. Select the **Installed** tab.
8. Select the radio button next to the languages to activate the keyboard language for that languages.


 Make sure to select the radio button for all the languages that you want a voice available for.

9. Select the  (Back) button to go back to TD Talk Settings.
10. Select the  (Close) button to exit TD Talk Settings.


## 5.3 Voice

Set the speaking voice of your keyboard languages here

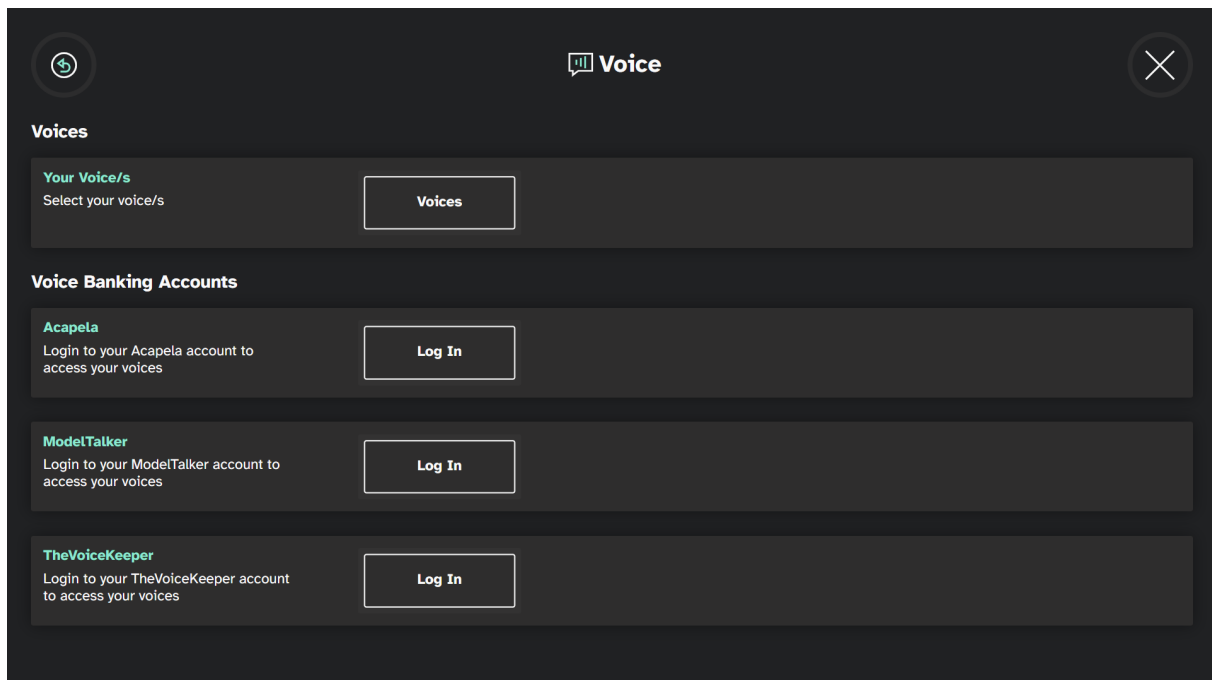



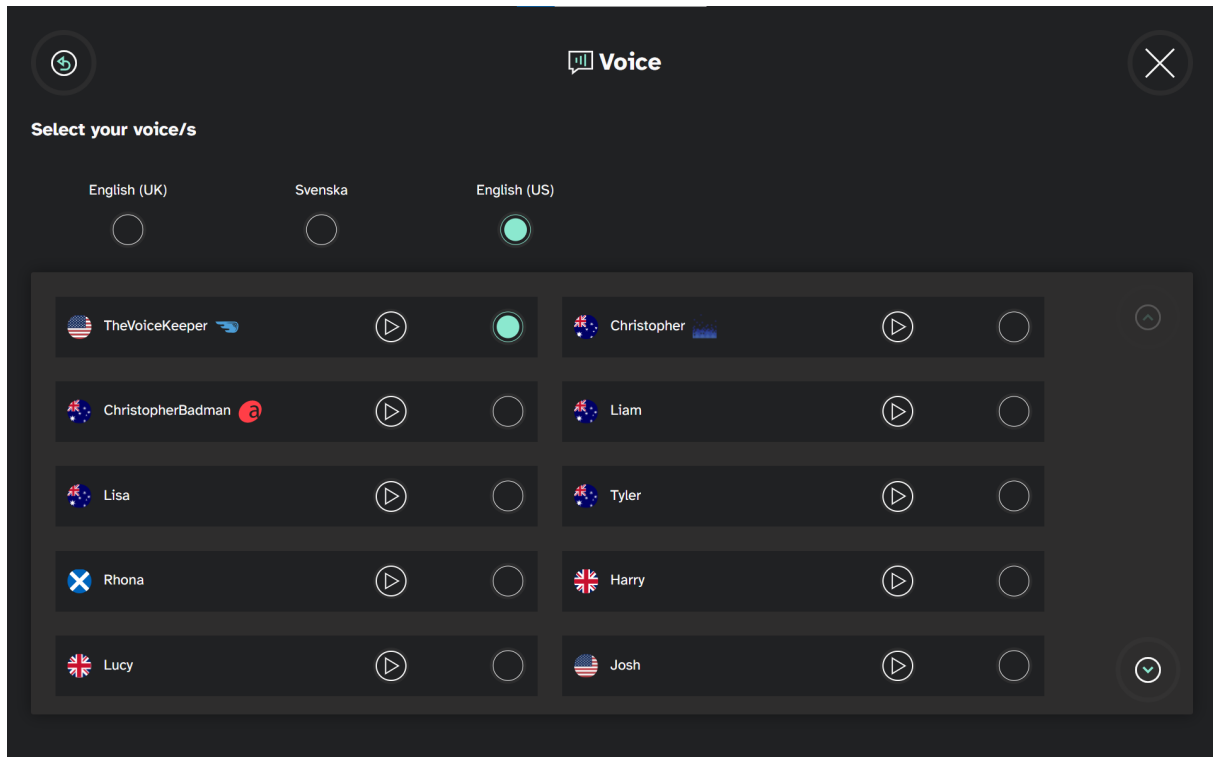

All languages installed are the shown in the top part of the screen. When the language has been selected, the available voices for that language will show in the list beneath the languages. To try out a voice, select the  next to the voice to try out the voice. If available.

### 5.3.1 Voices







#### 5.3.1.1 How do I select my Voice in TD Talk?

 The voices from the Voice Banking system will have the logo for the Voice Banking Service visible after the name of the voice.

- Acapela MyOwnVoice 
- ModelTalker 
- The Voice Keeper 

1. Fixate your gaze below the screen, in the middle of the eye tracker or the area of the eye tracker.

The  (More) button will become visible in the lower middle part of the screen.

2. Select the  (More) button to open the More.
3. Select the  (Settings) action to open the TD Talk Settings.
4. Look for the  (Voice) icon to find the correct settings card.
5. Select the **Open** button.
6. Look in the **Voice** section

7. Select the **Voices** button in **Your Voice/s**.
8. Select the radio button under the installed language you want to speak.





If the wanted language is missing, see 5.2.3.1 *How do I Add/Download a New Language/Keyboard Language in TD Talk?*, page 20 and 5.2.3.2 *How do I select and activate the Keyboard Languages to use/be available in TD Talk?*, page 21.

9. Select the voice to use in the list of voices available for the language to speak. Use the “Scroll up” and Scroll Down” buttons on the right to scroll through the list of voices available for the language.






To try out a voice, select the  next to the voice.

10. Select the  (Back) button to go back to TD Talk Settings.
11. Select the  (Close) button to exit TD Talk Settings.

### 5.3.1.2 How do I download a new Voice in TD Talk?


1. Fixate your gaze below the screen, in the middle of the eye tracker or the area of the eye tracker.

The  (More) button will become visible in the lower middle part of the screen.

2. Select the  (More) button to open the More.
3. Select the  (Settings) action to open the TD Talk Settings.
4. Look for the  (Voice) icon to find the correct settings card.
5. Select the **Open** button.
6. Look in the **Your Voice/s** section.
7. Select the radio button under the installed language you want to speak.





If the wanted language is missing, see 5.2.3.1 *How do I Add/Download a New Language/Keyboard Language in TD Talk?*, page 20 and 5.2.3.2 *How do I select and activate the Keyboard Languages to use/be available in TD Talk?*, page 21.

8. Select the  (Download) button next to the language to start the download of the new keyboard. Use the “Scroll up” and Scroll Down” buttons on the right to scroll through the list of voices available for the language.






To try out a voice, select the  next to the voice.

9. Select the **Download** button to start the download or the **Cancel** button to cancel the download.
10. Select the voice to use in the list of voices available for the language to speak.
11. Select the  (Back) button to go back to TD Talk Settings.
12. Select the  (Close) button to exit TD Talk Settings.


## 5.3.2 Voice Banking Accounts

TD Talk can connect to three different Voice Banking Services:


- Acapela MyOwnVoice 
- ModelTalker 
- The Voice Keeper 




The user will need an account with and the login credentials for one of these providers of voice banking services to be able to connect and use the personal synthetic voice.

### 5.3.2.1 How can I use my Voice from my Voice Banking Service?


 Internet access is needed for log in to the Voice Banking Service.


1. Fixate your gaze below the screen, in the middle of the eye tracker or the area of the eye tracker.



The  (More) button will become visible in the lower middle part of the screen.

2. Select the  (More) button to open the More.
3. Select the  (Settings) action to open the TD Talk Settings.
4. Look for the  (Voice) icon to find the correct settings card.
5. Select the **Open** button.
6. Select the **Log in** button in the section for your Voice Banking Service.
7. Enter the **User name** and **Password**.
8. Select the **Log In** button.

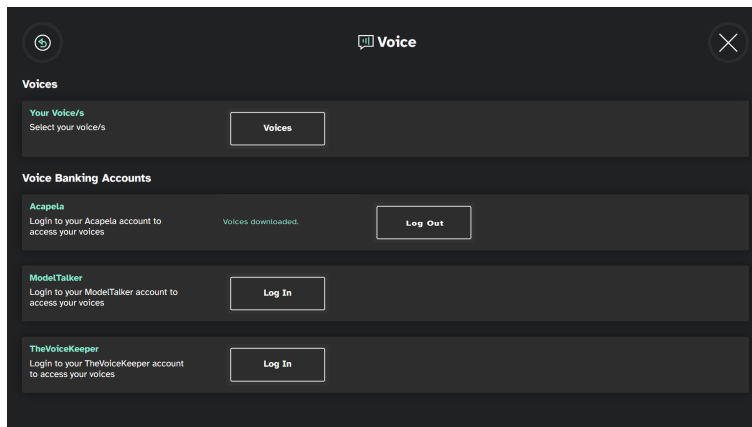The personal voice will be downloaded and automatically selected as the default voice for the user.




 This will be the case when personal synthetic voices in multiple languages are available

9. Select the  (Back) button to go back to TD Talk Settings.
10. Select the  (Close) button to exit TD Talk Settings.

## 5.4 Device



Only available on the TD Pilot.

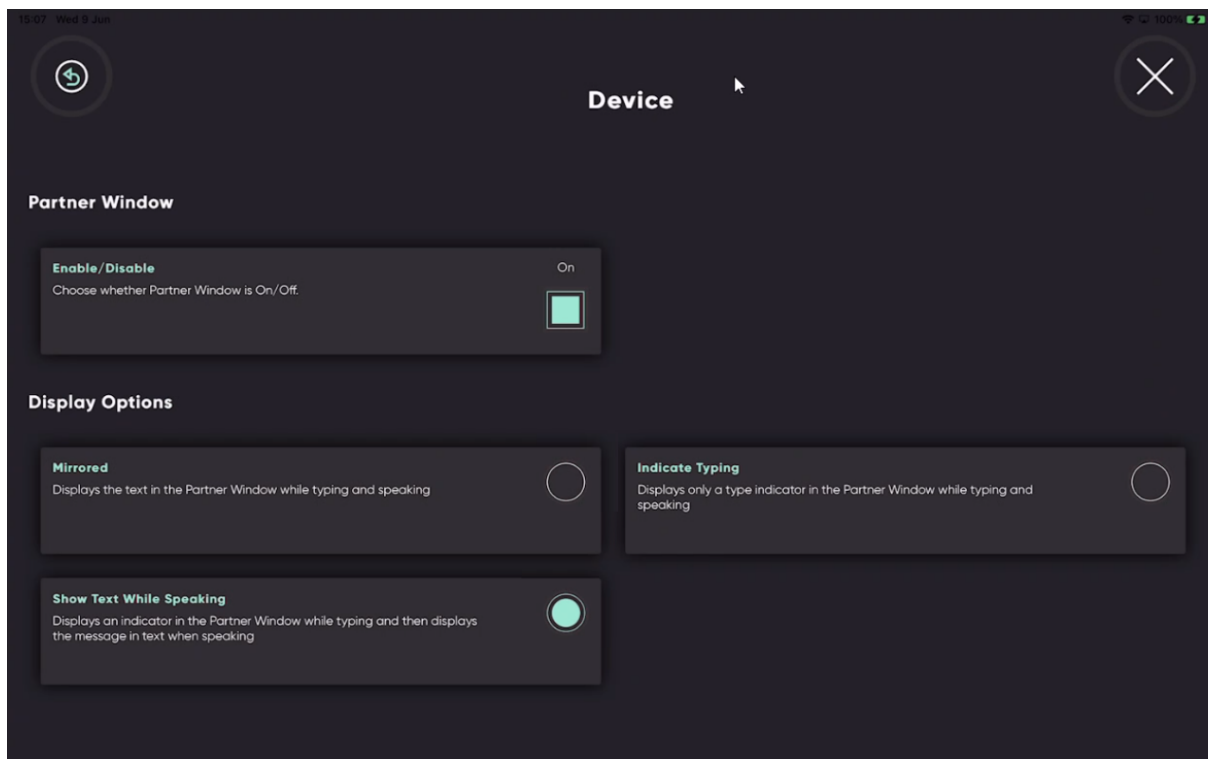

### 5.4.1 Partner Window

#### 5.4.1.1 How do I Enable/Disable the Partner Window in TD Talk?

1. Fixate your gaze below the screen, in the middle of the eye tracker or the area of the eye tracker.



The (More) button will become visible in the lower middle part of the screen.

2. Select the (More) button to open the More.
3. Select the (Settings) action to open the TD Talk Settings.



4. Look for the (Device) icon to find the correct settings card.
5. Select the **Open** button.
6. Look in the **Partner Window** section.
7. Toggle the **Enable/Disable** button for “Choose whether Partner Window is On/Off” to:

- On
- Off




8. Select the (Back) button to go back to TD Talk Settings.





9. Select the (Close) button to exit TD Talk Settings.


#### 5.4.1.2 How do I Set the Display Options for the Partner Window in TD Talk?

1. Fixate your gaze below the screen, in the middle of the eye tracker or the area of the eye tracker.

The  (More) button will become visible in the lower middle part of the screen.

2. Select the  (More) button to open the More.

3. Select the  (Settings) action to open the TD Talk Settings.


4. Look for the  (Device) icon to find the correct settings card.


5. Select the **Open** button.

6. Look in the **Display Options** section.


7. Select one of the following radio buttons choose the Display Option for the Partner Window:

- Mirrored — Displays the text in the Partner Window while typing and speaking.
- Show Text While Speaking — Displays an indicator in the Partner Window while typing and the displays the message in text when speaking.
- Indicate Typing — Displays only a type indicator in the Partner Window while typing and speaking.

8. Select the  (Back) button to go back to TD Talk Settings.

9. Select the  (Close) button to exit TD Talk Settings.



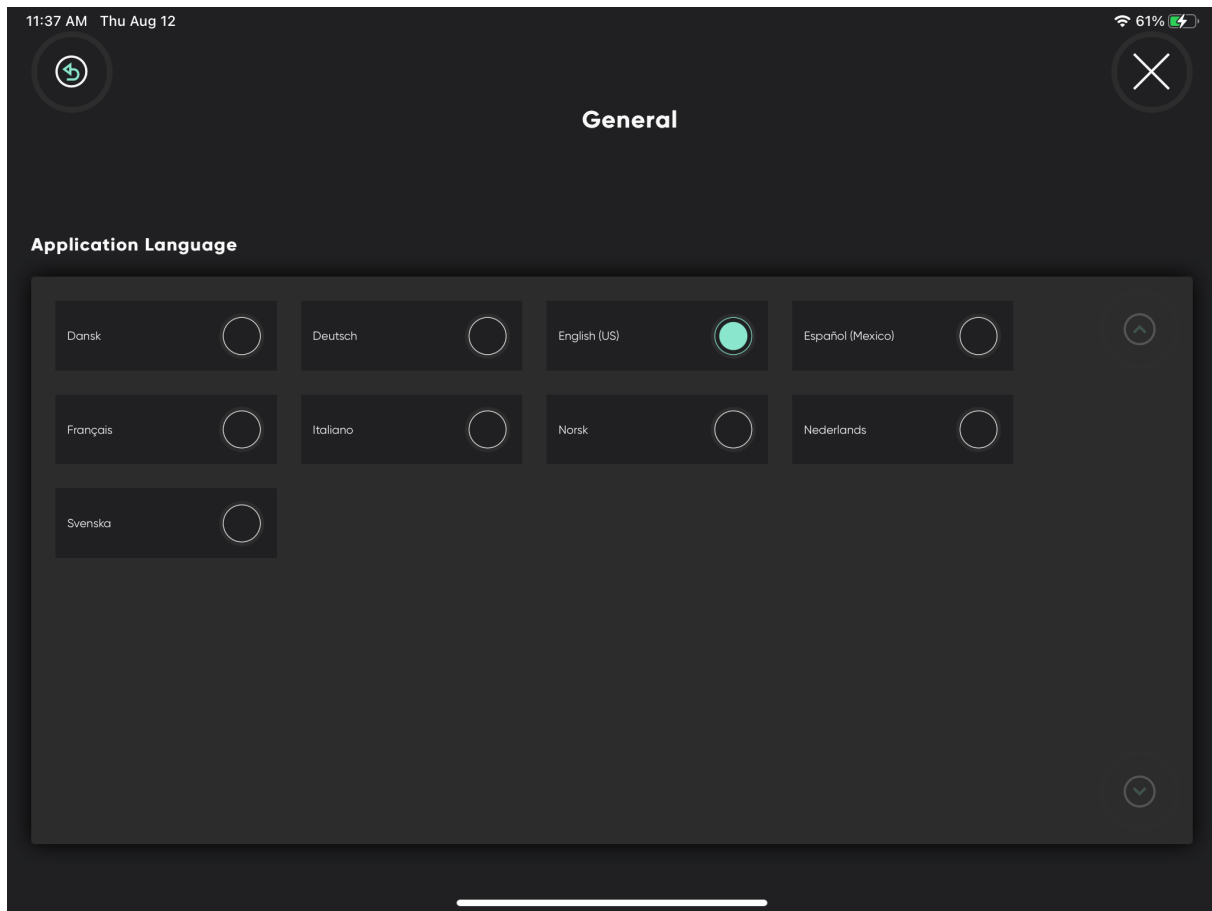



## 5.5 General



### 5.5.1 How do I set my Application language in TD Talk?

1. Fixate your gaze below the screen, in the middle of the eye tracker or the area of the eye tracker.

The  (More) button will become visible in the lower middle part of the screen.

2. Select the  (More) button to open the More.
3. Select the  (Settings) action to open the TD Talk Settings.
4. Select the  (Next Screen) button on the right-hand side.
5. Look for the  (General) icon to find the correct settings card.
6. Select the **Open** button.
7. Look in the **Application Language** section.
8. Select the radio button next to the language you want the application to be in.
9. Select the  (Back) button to go back to TD Talk Settings.
10. Select the  (Close) button to exit TD Talk Settings.

Copyright ©Tobii Dynavox AB (Publ). Not all products and services offered in each local market. Specifications are subject to change without prior notice. All trademarks are the property of their respective owners.

## Support for Your Tobii Dynavox Device

### Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: [www.TobiiDynavox.com/support-training](http://www.TobiiDynavox.com/support-training)

### Contact Your Solution Consultant or Reseller

For questions or problems with your product, contact your Tobii Dynavox solution consultant or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit [www.TobiiDynavox.com/contact](http://www.TobiiDynavox.com/contact)